

# **Biomedical Systems**

# Aerospray® Hematology Pro

Slide Stainer/Cytocentrifuge

**Series 2** 



## **SEE WHAT YOU NEED TO SEE**

### ALCOHOL BASED

The only slide stainer with built-in cytocentrifugation and fully customizable, programmed staining for a large variety of sample applications.



# **AEROSPRAY® TECHNOLOGY GIVES SUPERIOR STAINING RESULTS**

- Superior staining results in 7-12 minutes, allowing increased work flow in any lab
- Allows seamless staining with a minimal footprint
- Programming capabilities allow reliable and standardized staining and continuity across multiple labs within a network

■ Patented spray technology eliminates possible crosscontamination, with a fresh application of stain applied to every slide

 18-month shelf life for all reagents, providing flexibility with running small and large volumes of slides

Low maintenance









### **High-Quality Slides for Superior Smearing**

- Optimized for Aerospray Slide Stainers, Cytocentrifuges, and CellaVision® Digital Cell Morphology Analyzers
- High-quality white soda-lime glass enables smooth smears and even monolayers
- Blank or 45° clipped corner microscope slides

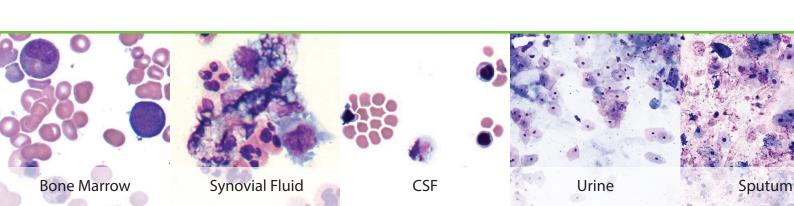


# PROGRAMMING CAPABILITIES DESIGNED FOR PREFERENCES - SEE WHAT YOU NEED TO SEE


**Option 1:** Select a default staining program

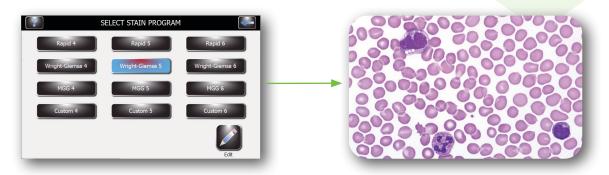

Staining Modes Include: Rapid Wright Giemsa, Wright Giemsa, and May Grünwald Giemsa


**Option 2:** Customize your own with easy programming capabilities

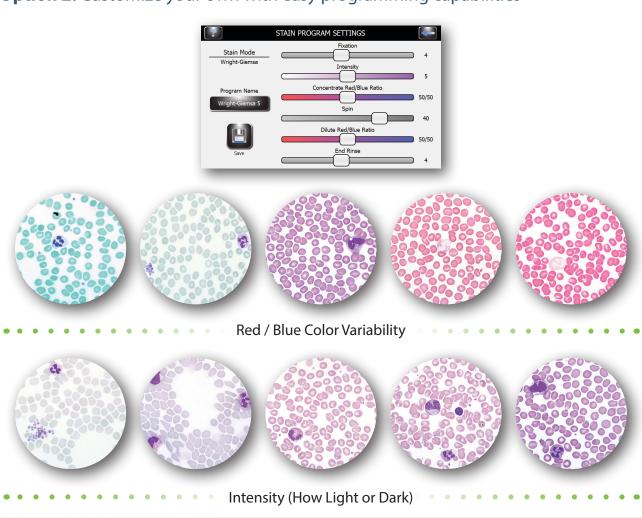

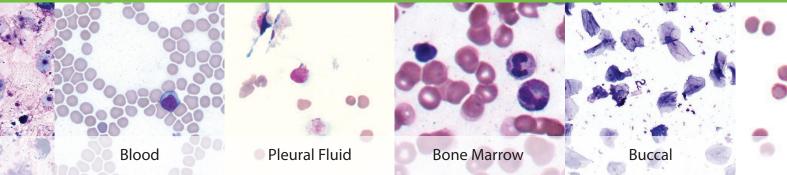


# BUILT-IN CYTOCENTRIFUGE FOR INCREASED CELL CONCENTRATION


WITHOUT CYTOPRO® ROTOR

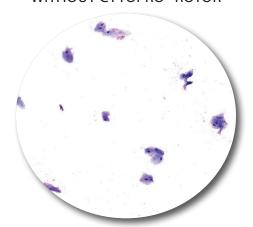

WITH CYTOPRO® ROTOR

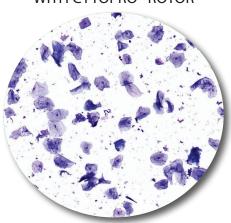


Exchange the slide carousel with the Cytopro® Rotor to transform the stainer into a superb cytocentrifuge.

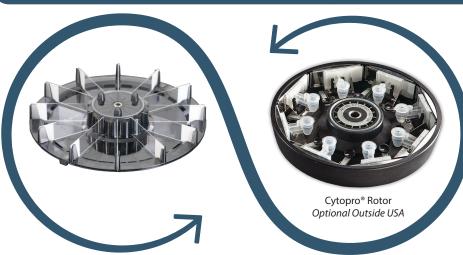



No purchase of a second instrument means consolidated maintenance and a smaller lab footprint.

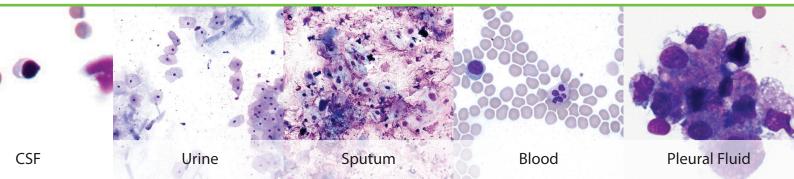

## **Aerospray® Hematology Pro Model 7152**

Slide Carousel Capacity	1 to 12 or 1 to 30 Slides, depending on carousel (carousel choice optional)
Cytocentrifuge Rotor Speed	100-2000 rpm (± 5%), user programmable
Dimensions (W x H x D)	57 cm x 25 cm x 54 cm (22" x 10" x 21")
Clearance Height (lid open)	58 cm (23")
Weight	16.3 kg (35.9 lb)
Electrical Requirements	100 to 240 VAC (±10%) @ 50-60 Hz, 200 watts maximum
Drain Connection	Outlet on rear panel
Safety	Lid interlock: lid must be closed for operation, and is locked down during carousel rotation
Reagent Level Detect	Level detection on all reagent lines
Waste Level Detect	Level detection in waste bottle (optional)

### Cytopro® Cytocentrifuge Rotor Model AC-160

Capacity	8 chambers and slides
Weight	1.1 kg (2.5 lb)
Dimensions (Dia. x H)	22.6 cm x 6.2 cm (8.9" x 2.4")

Catalog #	Description
MODEL 7152	Aerospray® Hematology Pro Slide Stainer/Cytocentrifuge
SS-029	Aerospray® Nozzle Cleaning Solution, 355mL
SS-029C	Aerospray® Nozzle Cleaning Solution Concentrate, 250mL
SS-048	Hematology Reagent D, Aerofix® Fixative, 500mL
SS-071A	Hematology Pro Reagent A, Buffer (pH 6.8), 1Gal
SS-071B	Hematology Pro Reagent B, Thiazin Stain, 500mL
SS-071C	Hematology Pro Reagent C, Eosin Stain, 500mL
SS-072A	Hematology Pro Reagent A, Buffer (pH 7.2), 1Gal
SS-148	Hematology Reagent D, Aerofix® Fixative Concentrate, 135mL
SS-171A	Hematology Pro Reagent A, Buffer (pH 6.8) Concentrate, 30mL
SS-171B2	Hematology Pro Reagent B, Thiazin Stain Concentrate, 200mL
SS-171C2	Hematology Pro Reagent C, Eosin Stain Concentrate, 220mL
SS-172A	Hematology Pro Reagent A, Buffer (pH 7.2) Concentrate, 30mL
SS-230	Aerospray® Stain Residue Solvent, 500mL
SS-MeOH	Aerospray® Reagent Grade Methanol, 500mL
AC-187	Barcode Scanner and Waste Level Detect Bundle
AC-188	12-Slide Carousel, Aerospray® Stainers
AC-189	30-Slide Carousel, Aerospray® Stainers
SS-280	Microscope Slides, Clipped Corner, Frosted, 1/2G, Uncoated
SS-287	Microscope Slides, Frosted, 1/2G, Uncoated

 ${\it Check for additions to catalog numbers made in your country.}$ 

Catalog #	Description
AC-160	Cytopro Rotor
SS-111	Cytopro Cytopad, fast, white, package of 100
SS-112	Cytopro Cytopad, slow, tan, package of 100
SS-113	Sample Chambers w/ fast, white Cytopads, caps, 48pk
SS-114	Sample Chambers w/ slow, tan Cytopads, caps, 48pk
SS-117	Microscope slides, 1/2G, uncoated, for Cytopro
SS-118	Microscope slides, 1/2G, coated, for Cytopro
SS-213	Dual Sample Chambers w/ fast, white pads, caps, 48
SS-214	Dual Sample Chambers w/ slow,tan pads,caps,pk/48
SS-217	Microscope Slides, 1/2G, uncoated, Cytopro Dual
SS-218	Microscope Slides, 1/2G, coated, Cytopro Dual
SS-232	Microscope slides, 1/2G, uncoated, for Cytopro Magnum
SS-233	Microscope slides, 1/2G, coated, for Cytopro
SS-234	Cytopro Magnum Sample Chambers, caps, 24pk



## **GLOBAL LEADER IN AEROSPRAY®** STAINING TECHNOLOGY AND CYTOCENTRIFUGATION



## "Deliver innovative diagnostic solutions that enable better medical decisions"

ELITechGroup has been in operation for over 50 years and distributes to over 100 countries. Proud to be the global leader in Aerospray staining technology and cytocentrifugation, offering the world's most reliable solutions in performance and reliability. We are confident that this automated technology will elevate the performance of any laboratory and facilitate superior results.

Join the thousands of laboratories utilizing the world's best staining technology and take your laboratory to a new level of efficiency and performance.











#### **WORLDWIDE OFFICES**

Please contact your sales representative for terms, conditions and product availability in your country.

#### Headquarters

Australia Belgium & Luxembourg France Italy New Zealand Germany

#### +33 1 41 45 07 10

T: +61 1800 815 098 T: +32 9 282 05 31 T: +33 4 83 36 10 82 T: +39 02 48 40 35 42

T: +64 800 555 611 T: +49 30 7809 588-0 Canada Serbia The Netherlands United States

T: +1 888 662 4925 T: +381 11 2467119 T: +31 313 430 500 T: +44 1442 869320 T: +1 435 752 6011

